D Gebrauchsinformation

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100 g DC Cal contains:
45 g Calciumhydroxide and
5 g Bariumsulfate

Indications

Supply of the caries-free, pulp-near dentine, treatment of caries profunda, indirect pulp-capping, direct pulp-capping, pulpotomy, inlay in root-canal.

Contra-indications

The use of DC Cal is contraindicated: opened pulp/-exposure in carious dentine, at ignited modified pulp, at super-sensitiveness against one of the ingredients.

Particular precautions:

DC Cal is always to be stored well closed. While application you'll have to put higher pressure on opened pulpa. Calcium hydroxide causes local irritation and corrosive effects of the skin, eyes and mucous membranes. Particles of calcium hydroxide in contact with skin or eyes must be mechanically removed by appropriate ways, then rinse immediately and thoroughly with water. Do not use this medical product after expiry date. Keep this medical product out of reach of children.

Interactions

The harmful effects of with acid mixed cements onto the pulp-/dentin-system can be reduced throughout calciumhydroxide-applications.

Method of use

The given quantity of DC Cal requires to the particular range of application:

- Caries profunda-treatment / indirect pulp capping:** DC Cal will pressureless be applied in a thin layer onto the bottom of cavity. Thereafter a temporary-/ or final filling will be set.
- Direct pulp capping:** After drying the treated surface, apply DC Cal and an adequate filling. A lot of pressure is required pressure when applying on open pulpa.
- Vital amputation:** DC Cal builds good conditions for healing of remaining pulp-stumps throughout the accepted local bounded etch-effects of the calciumhydroxide to the remaining pulp.
- Lining / pulp-protection:** The space-saving and radiopaque DC Cal builds a pulp-protection even in front-teeth-area under silicale- or composite fillings a thin, whitish insulating-layer against cement-acids or free monomer.
- Dentine-wound-bandage for grinded vital teeth:** With a brush or a wad of cotton wool DC Cal will be applied onto the preserved tooth-stump and carefully dried with warm air. Avoid contact with the gingiva. Beforehand dip the brush into cooked water / aqua dest. to dilute the paste.
- Medical inlay into infected root-canals:** Each root-canal should be air-bubble-free filled with DC Cal. Eventually the inlay should be relaced after 6 to 8 weeks.

Side effects

Side-effects with this medical product are extremely rare if it is properly prepared and applied. Should you encounter undesirable side-effects – even in case of doubt – then please contact us and give a precise description of the circumstances and symptoms.

Working instructions

Ensure that the processing instructions and precautionary measures described in the supplied product information are always strictly observed. Should the patient suffer undesirable reactions or any other unwanted incidents arise even though these instructions were observed, then please inform us and give a precise description of the circumstances.

Storage

Immediately after use the syringe top has to be protected with a cap. In case of direct application the tip has to be sterilised and disposed. Do not use hardened paste. Storage at temperatures of 15° C to 23° C.

Note

Applicator-tools (tips, brushes) are only determinate for single-use at only one patient.

Contents

Syringe with 2,0 ml paste + 3 disposable applications canules

- H315 Causes skin irritation.
H318 Causes serious eye damage.
H335 May cause respiratory irritation.

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