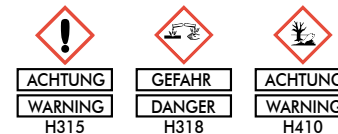
DC EugenolfreeCem

DC DentalCentral GmbH • Owiedenfeldstraße 6 • D-30559 Hannover • Germany • www.dental-central.de

Made in Germany
Stand 2022-REV-001

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Patientenzielgruppe

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Varianten

REF 702240 25,0 g Basis-Paste in Tube
25,0 g Katalysator-Paste in Tube

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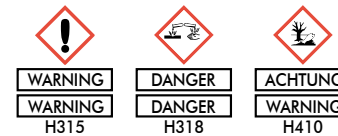
en Instructions for use

Patient target group

All those for whom DC EugenolfreeCem is required for treatment, except people who are allergic to the ingredients, here especially rosin.

Intended user

Dentist

Composition

Essential components: Resin acid and metal oxides

Qualitative data:

Paraffin, peanut oil, calcium hydroxide, magnesium oxide, hydroxyapatite, zinc oxide, ethanol (traces), fatty acid from coconut, rosin, silicon dioxide.

Quantitative data:

Calcium hydroxide 2%, rosin 45%, fatty acid 25%, zinc oxide 10%.

Intended purpose of the product

DC EugenolfreeCem is an easy-to-handle, eugenol-free cement that does not inhibit the curing of resin-based filling and luting materials. DC EugenolfreeCem is indicated for allergies to eugenol. DC EugenolfreeCem contains calcium hydroxide and hydroxyapatite to protect the teeth and preserve them.

Indications

DC EugenolfreeCem is a temporary bonding cement for crowns and bridges.

Contraindications

The material contains rosin. Do not use in patients with an allergy to rosin, peanut oil or other substances contained in the product.

Performance characteristics of the product

The product cures after mixing.

Application

Type of application:

DC EugenolfreeCem is processed at room temperature. Longer processing times result at low temperatures (cooled glass plate). The mixing ratio of the base (brown) and catalyst paste (white) is 2 to 1 in terms of volume (= 1 to 1 weight ratio). Therefore, the expressed strand of the base paste should be twice as long as the strand of the catalyst paste. Apply both pastes to a mixing block of non-absorbent material or a glass plate. Mix the paste for about 30 seconds until a homogeneous mass is obtained. Apply a thin layer to the inside of the temporary crown or bridge or wet die preparation. Never use on dry stumps, always moisten them first. DC EugenolfreeCem can be used for 1 to 2 minutes and cures in the mouth in 3 to 6 minutes. Remove excess material approx. 3 to 5 minutes after curing.

Additional notes:

Softer material and longer working times can be achieved by adding a little more DC EugenolfreeCem base paste (brown) or a small amount of Vaseline. In addition, moisture (damp stumps) makes it easier to remove later. Harder and faster setting material can be achieved by adding a little more DC EugenolfreeCem catalyst paste (white) or by mixing in zinc oxide. Such a harder material is better suited for multi- and large-surface cavities as well as temporary long-term closures (maximum 1 month). Residues of DC EugenolfreeCem can easily be removed from crowns by treating them with a cold spray.

Processing instructions:

The processing instructions and precautionary measures described in the instructions for use belonging to the product must be observed in any case. If, despite adherence to these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in any case, describing the accompanying circumstances as precisely as possible.

Warnings/residual risks

The product contains rosin. Persons who have an allergy to rosin should not come into contact with the product. Avoid contact of mixed cement with skin or mucous membrane. If contact does occur, remove the material with cotton wools soaked in alcohol and rinse with plenty of water. In case of eye contact, rinse with plenty of water with the eyelids open and consult an ophthalmologist.

Risks/side effects

Risks of overdose are not known. Adverse side effects are not known.

Interactions

Interaction of the product and its metabolites with other medical devices, medicinal products or other substances is not known.

Storage

DC EugenolfreeCem should be stored at room temperature. Close the tubes well after use. Do not interchange closures.

Minimum shelf life

Do not use DC EugenolfreeCem after the expiry date.

Variants

REF 702240 25,0 g base paste in tube
25,0 g catalyst paste in tube

Disposal

Uncured material should be disposed of as hazardous waste.

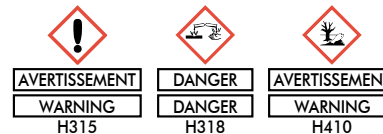
Safety instructions

H315 Causes skin irritation.
H318 Causes serious eye damage.
H410 Very toxic to aquatic life with long lasting effects.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

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