DC DentalCentral GmbH • Owiefenfeldstr. 6 • D-30559 Hannover • Germany

PRISMAN GmbH • Otto-Hahn-Ring 6-18 • D-64653 Lorsch • Germany • Tel.: +49 (0)6251 866980-0

UFI-Nr.: FM05-71U9-WT05-MPC9

DC Keramik Ätzgel

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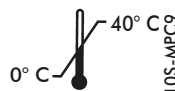
GB Instructions for use

Intended use:

DC Ceramic etching gel is an etching gel based on hydrofluoric acid and is used for extraoral etching of ceramic restorations.

Areas of application:

The adhesion between the glass ceramic surface and the luting composites is increased, by the production of a retentive etching pattern, for all-ceramic restorations such as inlays, onlays, veneers or crowns.

Application:

1. DC Ceramic etching gel is applied with a small disposable brush or plastic rod to the ceramic surfaces to be etched.
2. Recommended is an etching time of 60 sec.

3. After the application time, remove the acid residues entirely. To do this, intensively steam-blast the surface, spray it with water or clean it in an ultrasonic bath using distilled water. The etching of the ceramic surfaces is preferably carried out immediately before the insertion or after try-in. Avoid contamination of the etched surfaces (do not brush). However, if contamination occurs after etching, then the surface must be shortly etched again (approx. 10 sec). Dry well after etching, then continue to use adhesion promoters, etc. according to the manufacturers' instruction manual. Only for extraoral use! DC Ceramic etching gel must not be used for etching tooth substance in the mouth!

Content: 6 ml e

Safety instructions:

H310 *Fatal in contact with skin.*

H331 *Toxic if inhaled.*

H314 *Causes severe skin burns and eye damage.*

Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Special treatment (after skin contact: immediately treat with calcium gluconate solution or calcium gluconate gel). Dispose of contents/container in accordance with all local / regional / national and international regulations. Contains <5% hydrofluoric acid. For professional use only.

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DC DentalCentral GmbH • Owidenfeldstr. 6 • D-30559 Hannover • Germany