ACHTUNG
WARNING
H315 H411
H319

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Patientenzielgruppe

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en Instructions for use

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentist

Composition

Qualitative data:

Calcium sulphate, zinc oxide, ethyl cellulose

The product „DC Temporaryfill, pink“ also contains the following ingredients:
Indanthrene brilliant pink R, limonene and L-menthyl acetate.

Quantitative data:

60 % calcium sulphate, 15 % zinc oxide

Intended purpose of the product

For temporary fillings.

Indications

Temporary, short-term closure of dental cavities.

Contraindications

The product „DC Temporaryfill, pink“ contains the flavouring agent „limonene and L-menthyl acetate“. The product should not be used in the case of known hypersensitivities and allergies to these ingredients.

Performance characteristics of the product

Sealing compound, self-curing.

Specific precautions

None

Application

Apply the required amount of DC Temporaryfill into the moist cavity with a filling instrument. Hardening begins after a few minutes. In exposed areas, a 15-minute rest is recommended. Deep cavities do not need to be filled completely, thereby facilitating later removal. When closing inserts, ensure that the application is as free of pressure as possible.

The best way to remove the set DC Temporaryfill is to use a pointed, short and strong probe inserted between the dentin and the occlusion.

Warnings

Residual risks

The processing instructions and precautionary measures described in the instructions for use accompanying the product must always be observed. If, despite adherence to these instructions, an adverse reaction occurs in the patient or other adverse events occur, we request that you inform us in any case, describing the accompanying circumstances as precisely as possible.

Side effects

Adverse side effects are not known.

Risks in case of overdose are not known.

Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage

Close the container immediately after each material withdrawal. Shelf life of the closed packaging: see label. After opening for the first time, the preparation can be stored for at least 6 months. The Product should be stored between 0°C - 22°C

Minimum shelf life

Do not use DC Temporaryfill after the expiry date.

Note on DC Temporaryfill pink:

Colour deviations from pink do not represent a defect and have no influence on the function, usability and safety of the product.

Variants

REF 702250 2 glasses at 30.0 g, pink

REF 702251 2 glasses at 30.0 g, white

Disposal

Unused material can be disposed of with household waste.

Safety instructions

H315 causes skin irritation

H319 causes serious eye irritation

H411 toxic to aquatic organisms, with long-lasting effect

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

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