DC DENTAL CENTRAL

DC Tray-Adhesive

indigodental GmbH • Fahltkamp 5 • D-25421 Pinneberg • Germany

DC DentalCentral GmbH • Owiefeldstraße 6
D-30559 Hannover • Germany • www.dental-central.de

D Gebrauchsinformation

Produktbeschreibung

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GB Instructions for use

Product description

DC Tray-Adhesive is suitable for use on impression trays with minimal or no retention areas. It offers a secure adhesion of addition curing silicones to metal and plastic impression trays. When using a plastic impression tray, first check its compatibility with the tray adhesive.

Indications

All dental situations requiring final impressions e.g. tooth loss, caries and trauma.

Contraindications

Do not use the material where there are contact allergies or known allergies to any of the components.

Intended use

DC Tray-Adhesive offers a secure adhesion of addition curing silicones to metal and plastic impression trays.

Limitations of use

DC Tray-Adhesive should only be used with addition curing, silicone-based impression materials.

Intended users

Dentist

Patient target group

Persons receiving treatment as part of a dental procedure.

Recommended use

1. The impression tray must be clean, grease free and dry.
2. Brush the inside of the tray with DC Tray adhesive so that there is a uniform, thin film.
3. Allow the adhesive to react for 3 minutes to ensure a good bond to the tray.
4. Load impression material into the tray within 30 minutes so that the adhesive produces its full bond strength.

Notes for use

- Highly flammable
- Do not inhale
- Do not swallow
- Avoid contact with skin and eyes
- Avoid skin contact with the fluid. In case of accidental contact, rinse with water and soap. In case of contact with the eyes, rinse immediately with lots of water and consult a physician.

Interactions

When using a plastic impression tray, first check its compatibility with the tray adhesive.

Residual risks / side effects

There are no known side effects to date. The residual risk of hypersensitivity to components in the material cannot be ruled out.

Warnings / precautions

- Keep out of the reach of children!
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Follow the manufacturer's instructions for other products that are used with the material/materials.
- Serious incidents involving this product must be reported to the manufacturer and to the responsible registration authorities.

Performance characteristics of the device

Viscosity: 0.5 - 10 [Pa·s], secure bonding

Storage / disposal

- Store at room temperature (15 – 25 °C / 59 – 77 °F).
- Close bottles immediately after use to prevent contamination as well as evaporation of the solvent.
- Do not use after the expiration date.
- Disposal must comply with national regulations.

Composition

Isoalkane, siloxane copolymer, modified resin, additives. Contains hexane

Packaging

REF 701320 1 bottle à 10 ml